

Distributed by



OPERACE

by PB Swiss Tools

SCREW REMOVAL SYSTEM SURGICAL TECHNIQUE



 100% Made in Switzerland

PB SWISS TOOLS



TABLE OF CONTENTS

OPERACE SCREW REMOVAL SYSTEM	Introduction	2
APPLICATIONS	Applications	3
PRODUCT OVERVIEW	Sterile single-use inserts	4
	Instruments for multiple use	5
	OPERACE Set Contents	6
SURGICAL TECHNIQUE	Using OPERACE correctly	9
	Procedure 1 Removal of undamaged intact screws via screwdriver shafts	10
	Procedure 2 Removal of screws with a damaged screw recess	12
	Procedure 3 Removal of locking head screws stuck in the plate which cannot be removed by any of the above procedures	13
	Procedure 4a Removal of broken, projecting screws or screws processed according to procedure 3	14
	Procedure 4b Removal of buried broken screw shafts	15
PRODUCT LINE	OPERACE Set List Codes - MINI	16
	OPERACE Set List Codes - SMALL	17
	OPERACE Set List Codes - LARGE	18
IMPORTANT INSTRUCTIONS	Important Instructions General Instructions Instructions for single-use products Instructions for reprocessable products	19
	Interpretation of Symbols	20
REPROCESSING	Reprocessing	21

OPERACE SCREW REMOVAL SYSTEM

The OPERACE Screw Removal System is used to remove implant screws made of titanium, titanium alloys and stainless steel – particularly damaged screws.

OPERACE offers a solution for the extraction of implant screws through the combination of durable handles and a range of single-use sterile inserts.

The system is organised into three colour-coded sets based on screw diameter:

- Instrument set MINI for mini fragment, usually for screw diameters 0.9–2.0, yellow colour-coding
- Instrument set SMALL for small fragment, usually for screw diameters 2.3–4.0, blue colour-coding
- Instrument set LARGE for large fragment, usually for screw diameters 4.2–8.0, green colour-coding










APPLICATIONS

For the removal of undamaged, damaged, and broken screws made of titanium, titanium alloys and stainless, in particular:

- Angular stable screws
- Cortex screws
- Cancellous bone screws
- Shaft screws
- Cannulated Screws
- Locking screws
- Locking bolts

Screwdriver Inserts

Hex	Torx®/ Stardrive®	Square/ Robertson	Cruciform	Slotted	Phillips
Screw head	Screw head	Screw head	Screw head	Screw head	Screw head
 HEX	 T	 SQR	 CR	 SL	 PH
Nominal tool size	Nominal tool size	Nominal tool size	Nominal tool size	Nominal tool size	Nominal tool size
1.3 mm, 1.5 mm, 1.8 mm, 2.0 mm, 2.5 mm, 3.0 mm, 3.5 mm, 4.0 mm, 5.0 mm	T4, T5, T6, T7, T8, T9, T15, T20, T25, T30, T40	0.7 mm, 1.0 mm, 1.2 mm, 1.5 mm, 1.8 mm, 2.2 mm, 2.3 mm	0.3 mm, 0.4 mm, 0.5 mm, 0.6 mm, 0.8 mm, 1.0 mm	0.3 mm, 0.4 mm, 0.5 mm, 0.6 mm, 0.8 mm, 1.0 mm	PH1

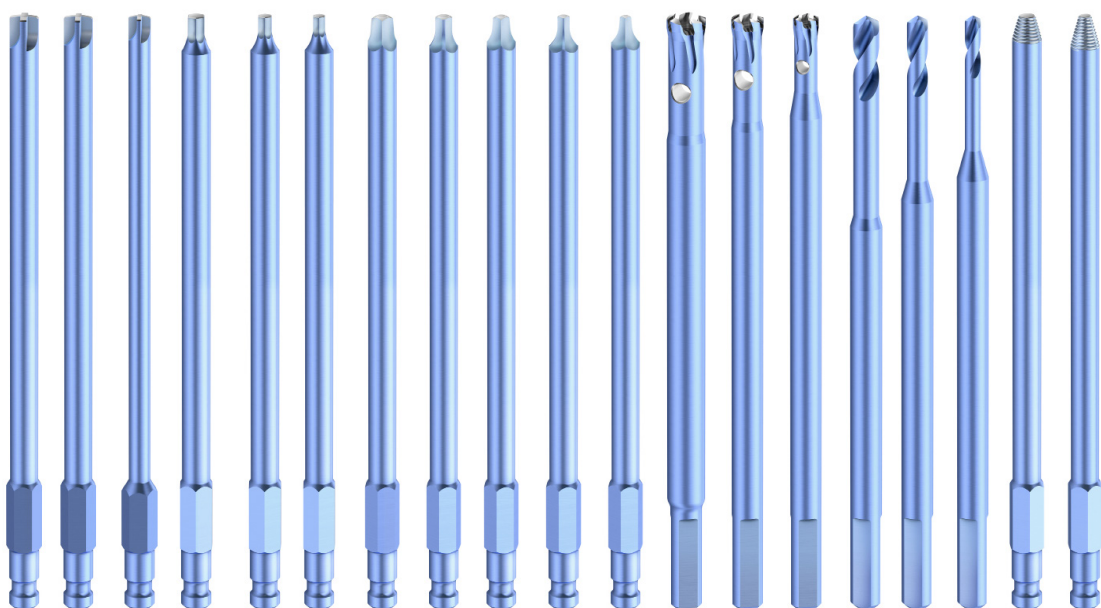

OPERACE Screw Removal System for Small Fragment screws

STERILE SINGLE-USE INSERTS

Every OPERACE instrument set contains single use inserts which can be used with OPERACE handles, cross-handles and extensions or power tools with adapters.

The single use inserts, extraction screws, drills and reamers are supplied in sterile form in double blister packaging and are intended for single use only.



OPERACE Screwdriver inserts
Removal of intact screws with the following drives:

- Hexagonal socket (Hex)
- Torx®/Stardrive®
- Square socket (Robertson)
- Cruciform
- Slotted
- Phillips



OPERACE Extraction Screws
With left-hand thread, and available in a range of sizes, for the removal of screws with a damaged hexagonal socket, Torx®/Stardrive® or square socket drive.



OPERACE Extraction Drill Bit
Removal of stuck locking-head screws. The screw head is separated from the screw shaft by drilling, enabling the plate to be removed with the separated screw head.



OPERACE Extraction Reamer
Left-turning, removal of drilled or broken screws.
Three operations in one: reaming, gripping, extracting. Various sizes matched to their thread diameter.

Notes

- Sterile-packed, for single use. Inserts may not be reprocessed.
- After determining the correct insert size, remove the sterile insert from the packaging using aseptic surgical techniques.
- Do not resterilise and do not reuse if the packaging is damaged or torn.
- Unused components may not be resterilised.

INSTRUMENTS FOR MULTIPLE USE

The OPERACE handles, cross-handles, extensions and AO adapters are supplied in non-sterile form. They are intended for multiple use (reprocessing).

OPERACE handle

- For rotation and universal use
- The ergonomic groove shape rests comfortably in the hand and allows high torques to be transmitted
- The large rounded end of the handle enables considerable axial pressure to be applied

OPERACE cross-handle

- For application of extremely high torques
- Optimally adjusted to fit the hand perfectly
- Due to the slant angle, the wrist remains straight and is protected when using the screwdriver with high force



OPERACE

Handles with quick coupling

The handles and cross-handles with quick coupling are designed to accept all OPERACE screwdriver inserts, extraction screws and extensions.

OPERACE

Couplings with AO/Synthes adapter for power tools

Couplings with an AO/Synthes adapter can be used as a link between power tools and all OPERACE screwdriver inserts, extraction screws and extensions.

Important

- The handles, extensions and couplings are intended for the loosening and unscrewing of implant screws.
- Blows and bending loads should be avoided as they can lead to instrument damage or breakage.
- The reprocessing must be carried out as described in the section on reprocessing.

OPERACE SET CONTENTS

Three set sizes with practical colour-coding


The colour-coding at the end of the handles corresponds to the colour-coded size classification of the inserts (MINI, SMALL or LARGE).

The purpose of the size specific handles for the screwdriver inserts i.e. the sizes MINI, SMALL and LARGE, is to ensure that the achievable manual torque of the handle is matched to the torque appropriate to the screw size.

Instrument set MINI for mini fragment
Yellow colour-coding
Screw diameters 0.9–2.0 mm
Quick coupling with one ring




Instrument set SMALL for small fragment
Blue colour-coding
Screw diameters 2.3–4.0 mm
Quick coupling with two rings

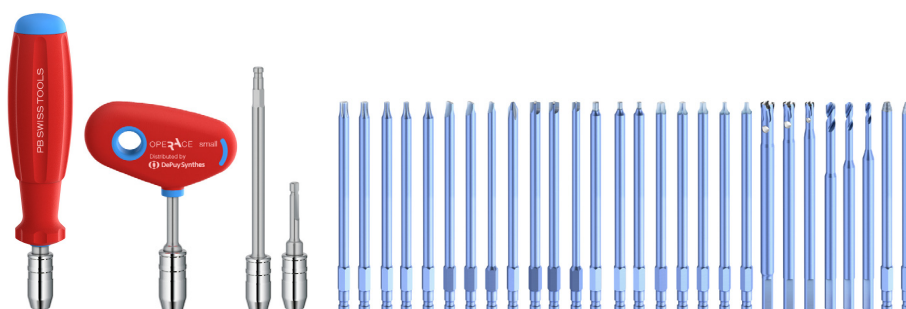


Instrument set LARGE for large fragment
Green colour-coding
Screw diameters 4.2–8.0 mm
Quick coupling with three rings



Insert insertion/removal

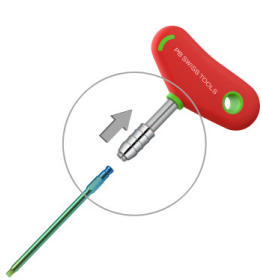

Pull back quick coupling

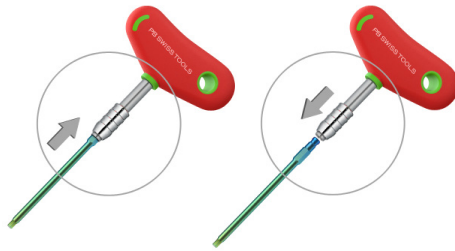

Insert or remove insert



Release quick coupling to lock in place

Note

- Check for correct engagement by pulling on the screwdriver insert

Storage container

Four-drawer storage container holding all single-use sterile inserts and processed (sterile) handles/instruments.



SURGICAL TECHNIQUE



USING OPERACE CORRECTLY

Extracting screws requires experience and expertise. The OPERACE Screw Removal System may be used only by trained operators.

The range of extraction products can be used for the following four procedures:

1. Removal of intact screws.
2. Removal of screws with a damaged screw recess.
3. Removal of locking head screws stuck in the plate.
4. Removal of broken screws and of screws processed according to procedure 3.



Important

- The correct instrument sizes to remove screws must be carefully chosen to facilitate screw extraction and to reduce the risk of further screw damage during the procedure.

PROCEDURE 1

Removal of intact screws that can be loosened by hand

Instruments used

- Handle or cross-handle of the appropriate size.
- Screwdriver insert, sterile-packed, single-use.

Option

- Extensions for the removal of deeply inserted screws
- Adapter with AO coupling for the use of instruments with power tools.

Note

- Start with the handle.
- If the screw cannot be rotated, use the cross-handle.

Procedure




1. Select screwdriver insert of the appropriate size and shape according to table 1.
2. Carefully clean screw recess.
3. Lock screwdriver insert in the handle coupling.
4. Insert the screwdriver insert into the screw recess.
5. Loosen screw manually by turning counterclockwise.

Option

- After it is loosened, remove the screw with the adapter with AO coupling and power tool

Important

- Use slot screwdriver inserts manually only
- Ensure that the screwdriver insert is fully inserted into the screw recess, otherwise the insert may spin in the recess.

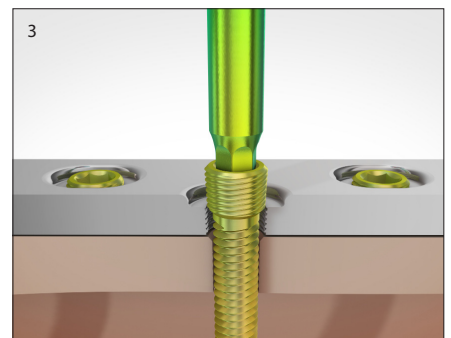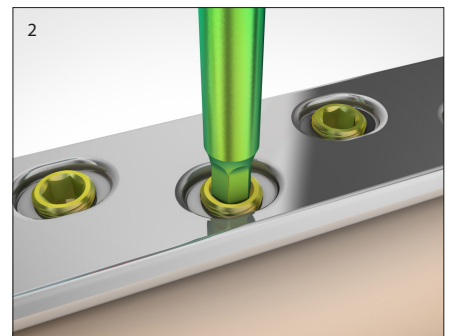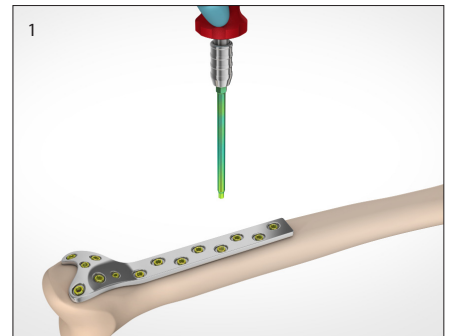

PROCEDURE 2

Removal of screws with a damaged screw recess

Instruments used

- Handle or cross-handle of the appropriate size.
- Extraction screw of the appropriate size, sterile-packed, single-use.

Procedure

1. Select extraction screw of the appropriate size matched to the screw recess size according to table 2.
2. Lock extraction screw in the handle coupling.
3. Start turning the extraction screw to the left, counterclockwise, in the same axis as the screw to be removed.
4. Continue turning, applying constant pressure, until the conical left-handed thread is securely seated in the damaged recess and until sufficient torque is applied to unscrew the screw.
5. Then unscrew the screw, turning to the left.

Important

- Turn to the left, counterclockwise.
- Use extraction screws only for removing screws with a damaged drive recess.
- Use extraction screws manually only.

Note

- If the extraction screw spins in the screw recess, an attempt can be made to drill the screw recess with the corresponding extraction drill bit to anchor the extraction screw more deeply.

Note

- For HEX 5.0 mm, T30 and T40 screw drives, a slightly deeper hole can be drilled in the recess with extraction drill bit 4.0.

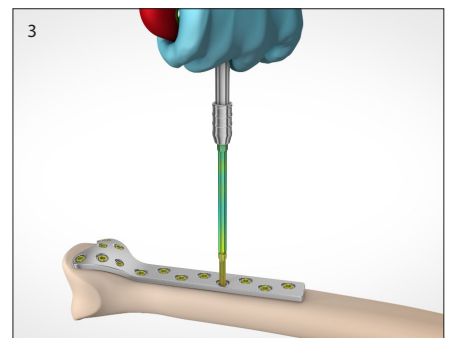
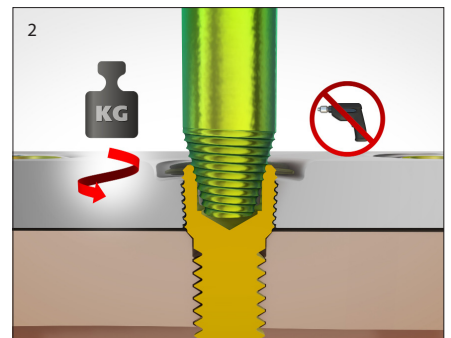
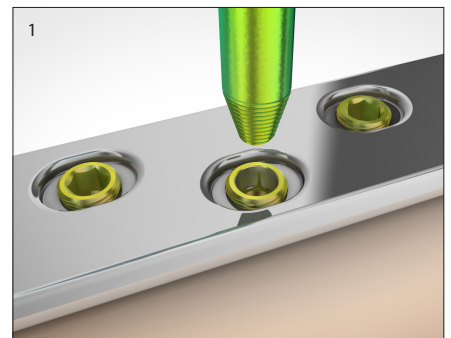


Table 2
Screw recess size

				OPERACE REF				
Hex 1.5	T6, T7	SQR 1.0, 1.2, 1.5	80018	Extraction screw # 1.6				
				Hex 1.8, 2.0	T8, T9	SQR 1.8	80170	Extraction screw # 2.0
				Hex 2.5, 3.0	T15	SQR 1.8, 2.2	80020	Extraction screw # 2.6
				Hex 3.5, 4.0	T20, T25	SQR 2.3	80022	Extraction screw # 3.5

PROCEDURE 3

Removal of locking head screws stuck in the plate which cannot be removed by any of the above procedures

Instruments used

- Power tool with Jacobs chuck.
- Extraction drill bit of the appropriate size, sterile-packed, single-use.

Procedure

1. Select extraction drill bit of the appropriate size according to table 3.
2. Position the drill bit in the screw recess and start turning the drill bit matching the screw diameter to the right, clockwise, in the same axis as the screw to be removed and applying very little pressure.
3. Only drill down until the underside of the implant plate is reached. This either loosens the screw head from the shaft or else weakens it sufficiently to cause it to break off when the plate is lifted.
4. Remove the plate.
5. Remove the screws according to procedure 4.

Important

- Turn to the right, clockwise.
- Apply the drill bit while it is already rotating, not while it is stationary.
- Apply very little pressure initially, only using the weight of the power tool, and start with a slow speed.
- Cool the bit and aspirate drill chips during the procedure. In order to avoid overheating, relieve the pressure on the drill from time to time.
- When the bit has reached the base of the recess, increase the pressure on the screw in order to obtain good chip formation.
- Do not use the bit to drill in bone or drill out broken screwdriver tips.




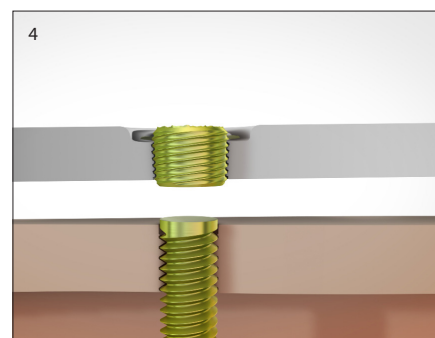


Table 3
Screw diameter in mm

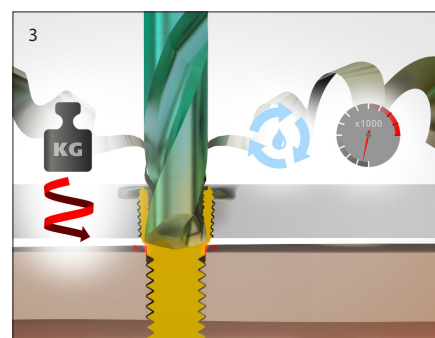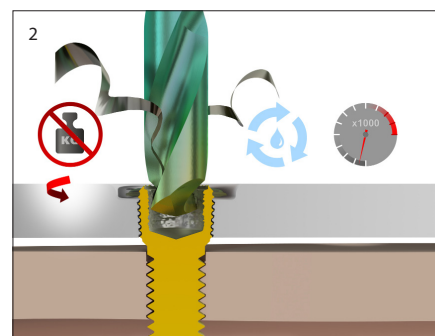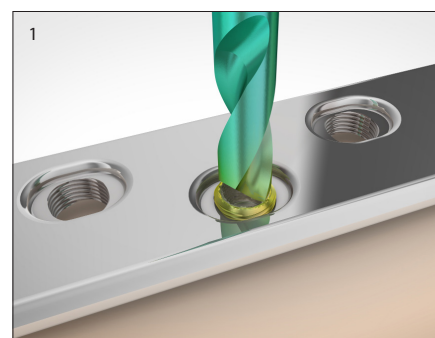

∅	OPERACE REF	
1.0–1.5	80023	Extraction drill bit 1.5
1.5–2.0	80123	Extraction drill bit 2.0
2.3–2.5	80024	Extraction drill bit 2.5
2.6–3.2	80025	Extraction drill bit 3.2
3.4–4.0	80019	Extraction drill bit 4.0
4.2–5.0	80026	Extraction drill bit 5.0
5.3–6.5	80027	Extraction drill bit 6.5
7.0–7.5	80021	Extraction drill bit 7.5

PROCEDURE 4A

Removal of broken, projecting screws or screws processed according to procedure 3

Instruments used

- Standard T-handle with Jacobs chuck or
- Power tool with Jacobs chuck.
- Extraction reamer of the appropriate size, sterile-packed, single-use.

Procedure

1. Select extraction reamer of the appropriate size matched to the screw recess according to table 4.
2. Locate the stationary extraction reamer and turn in the direction of the screw axis to ream over the screw shaft remaining in the bone. Apply slight pressure initially, turning to the left, counterclockwise.
3. As soon as the reamer grips the screw shaft, continue reaming with increased pressure until the conical left-handed thread is securely seated on the screw shaft.
4. When unscrewing the reamer do not relieve the pressure, but maintain the constant axial pressure and direction of rotation.

Note






- A manual procedure with a T-handle and Jacobs chuck is preferred.

Important

- Turn to the left, counterclockwise.
When unscrewing the reamer do not relieve the pressure, but maintain the constant axial pressure and direction of rotation.
- If a power tool is used, it should be kept at a very low speed.
- Cool the reamer and aspirate the drill chips during the procedure.

Table 4
Screw diameter in mm

Ø	OPERACE REF	
1.5–2.0	80009	Extraction reamer 2.0
2.3–2.5	80028	Extraction reamer 2.5
2.6–3.2	80029	Extraction reamer 3.2
3.4–4.0	80030	Extraction reamer 4.0
4.2–4.5	80033	Extraction reamer 4.5
4.7–5.8	80035	Extraction reamer 5.8
6.0–7.5	80015	Extraction reamer 7.5

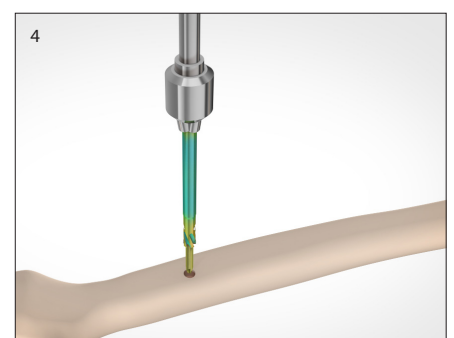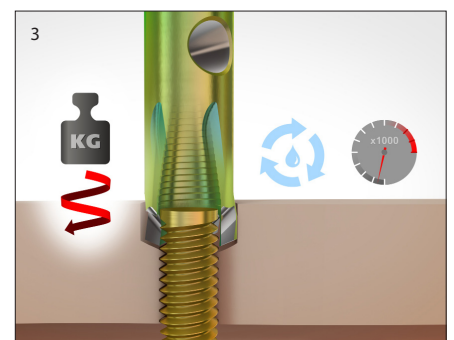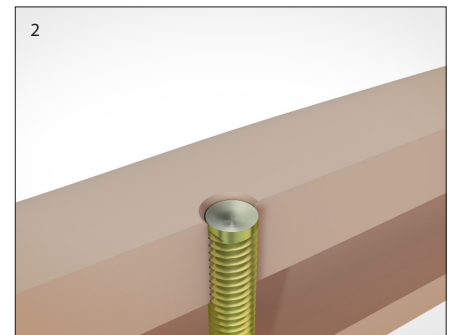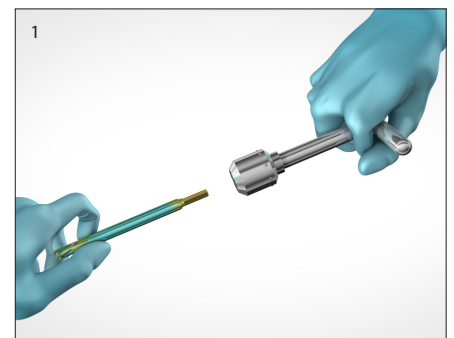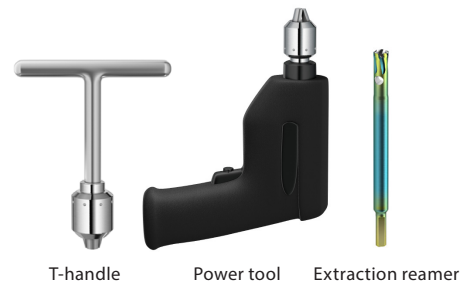

PROCEDURE 4B

Removal of buried broken screw shafts





Instruments used

- Standard T-handle with Jacobs chuck or
- Power tool with Jacobs chuck.
- Extraction reamer of the appropriate size, sterile-packed, single-use.




Procedure

1. Select extraction reamer of the appropriate size matched to the screw recess according to table 4.
2. Locate the stationary extraction reamer perpendicular to the bone so that the teeth rest on the bone as evenly as possible. Apply slight pressure initially, turning to the left, counterclockwise.
3. As soon as the reamer grips, ream in the direction of the screw axis under image intensifier control until it has been guided through the screw shaft.
4. Remove the reamer from the bone at regular intervals and free it of accumulated bone material.
5. Continue reaming with increased pressure until the conical left-handed thread is securely seated on the screw shaft.
6. When unscrewing the reamer do not relieve the pressure, but maintain the constant axial pressure and direction of rotation.

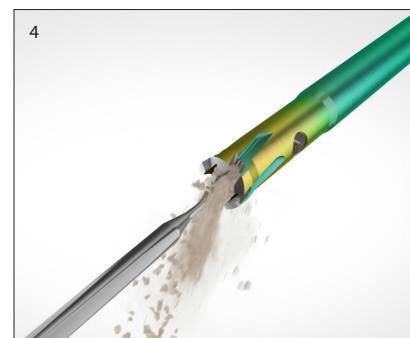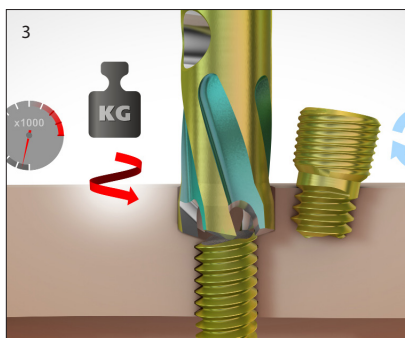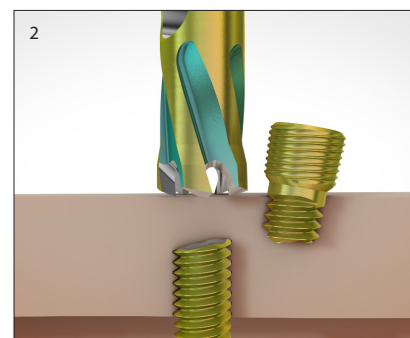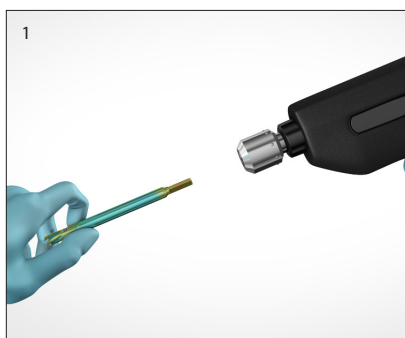


Important

- Turn to the left, counterclockwise.
- When unscrewing the reamer do not relieve the pressure, but maintain the constant axial pressure and direction of rotation.
- If a power tool is used, it should be kept at a very low speed. If the screw grips, a manual procedure with a T-handle and Jacobs chuck is preferred.

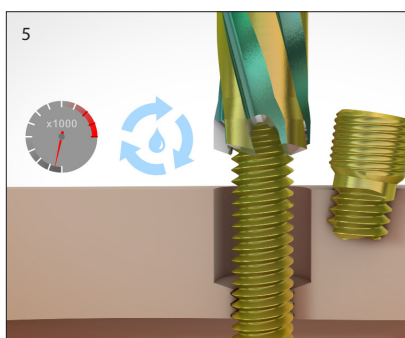

Note

Since the external diameter of *short-threaded cancellous bone screws* is smaller in the shaft than in the thread, smaller extraction reamers can be used if a break occurs in the shaft area. Although a reamer diameter smaller than that shown in the table 4 is used for this purpose, at least 10 mm of shaft without thread must be present.

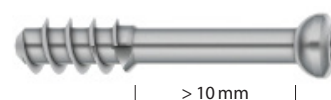













OPERACE SET LIST CODES - MINI



Reprocessable instruments

80001 Handle MINI

Sterile single-use inserts

		80196 Screwdriver insert for hexagonal socket screws, Hex 1.3
		80046 Screwdriver insert for hexagonal socket screws, Hex 1.5
		80069 Screwdriver insert for screws Torx® and Stardrive®, T4
		80070 Screwdriver insert for screws Torx® and Stardrive®, T5
		80071 Screwdriver insert for screws Torx® and Stardrive®, T6
		80055 Screwdriver insert for cruciform recess screws, Cr 0.3
		80056 Screwdriver insert for cruciform recess screws, Cr 0.4
		80060 Screwdriver insert for cruciform recess screws, Cr 0.5
		80036 Screwdriver insert for square socket screws, Sqr 0.7
		80037 Screwdriver insert for square socket screws, Sqr 1.0
		80061 Screwdriver insert for slotted screws, SI 0.3
		80062 Screwdriver insert for slotted screws, SI 0.4
		80065 Screwdriver insert for slotted screws, SI 0.5
		80018 Extraction screw, 1.6
		80023 Extraction drill bit, 1.5 f/ screws Ø 1.0 – 1.5
		80123 Extraction drill bit, 2.0 f/ screws Ø 1.5 – 2.0
		80009 Extraction reamer, 2.0 f/ screws Ø 1.4 – 2.0

OPERACE SET LIST CODES - SMALL



Reprocessable instruments	
	80002 Handle SMALL
	80004 Cross handle SMALL
	80010 Pick-up coupling, SMALL, with AO/Synthes adapter
Sterile single-use inserts	
	80051 Screwdriver insert for hexagonal socket screws, Hex 1.8
	80054 Screwdriver insert for hexagonal socket screws, Hex 2.0
	80048 Screwdriver insert for hexagonal socket screws, Hex 2.5
	80178 Screwdriver insert for screws Torx® and Stardrive®, T7
	80073 Screwdriver insert for screws Torx® and Stardrive®, T8
	80198 Screwdriver insert for screws Torx® and Stardrive®, T9
	80075 Screwdriver insert for screws Torx® and Stardrive®, T15
	80077 Screwdriver insert for screws Torx® and Stardrive®, T20
	80057 Screwdriver insert for cruciform recess screws, Cr 0.6
	80058 Screwdriver insert for cruciform recess screws, Cr 0.8
	80059 Screwdriver insert for cruciform recess screws, Cr 1.0
	80086 Screwdriver insert for Phillips screw, Ph 1
	80044 Screwdriver insert for square socket screws, Sqr 1.2
	80194 Screwdriver insert for square socket screws, Sqr 1.5
	80039 Screwdriver insert for square socket screws, Sqr 1.8
	80195 Screwdriver insert for square socket screws, Sqr 2.2
	80041 Screwdriver insert for square socket screws, Sqr 2.3
	80063 Screwdriver insert for slotted screws, SI 0.6
	80064 Screwdriver insert for slotted screws, SI 0.8
	80066 Screwdriver insert for slotted screws, SI 1.0
	80170 Extraction screw, 2.0
	80020 Extraction screw, 2.6
	80024 Extraction drill bit, 2.5 f/ screws Ø 2.3–2.5
	80025 Extraction drill bit, 3.2 f/ screws Ø 2.6–3.2
	80019 Extraction drill bit, 4.0 f/ screws Ø 3.4–4.0
	80028 Extraction reamer, 2.5 f/ screws Ø 2.3–2.5
	80029 Extraction reamer, 3.2 f/ screws Ø 2.6–3.2
	80030 Extraction reamer, 4.0 f/ screws Ø 3.4–4.0









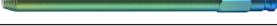








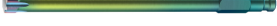

OPERACE SET LIST CODES - LARGE



Reprocessable instruments

80003	Handle LARGE
80005	Cross handle LARGE
80011	Pick-up coupling, LARGE, with AO/Synthes adapter

Sterile single-use inserts

		80197 Screwdriver insert for hexagonal socket screws, Hex 3.0
		80050 Screwdriver insert for hexagonal socket screws, Hex 3.5
		80052 Screwdriver insert for hexagonal socket screws, Hex 4.0
		80168 Screwdriver insert for hexagonal socket screws, Hex 5.0
		80079 Screwdriver insert for screws Torx® and Stardrive®, T25
		80081 Screwdriver insert for screws Torx® and Stardrive®, T30
		80083 Screwdriver insert for screws Torx® and Stardrive®, T40
		80022 Extraction screw, 3.5
		80026 Extraction drill bit, 5.0 f/ screws Ø 4.2 – 5.0
		80027 Extraction drill bit, 6.5 f/ screws Ø 5.3 – 6.5
		80021 Extraction drill bit, 7.5 f/ screws Ø 7.0 – 7.5
		80033 Extraction reamer, 4.5 f/ screws Ø 4.2 – 4.5
		80035 Extraction reamer, 5.8 f/ screws Ø 4.7 – 5.8
		80015 Extraction reamer, 7.5 f/ screws Ø 6.0 – 7.5

IMPORTANT INSTRUCTIONS

GENERAL INSTRUCTIONS

Disposal

Used single-use inserts must be disposed of correctly.

Return

Surgical instruments or sterile inserts may be returned only after consulting the distributor. All returned products must be decontaminated or sealed in the unopened original packaging. Unopened products with an exceeded expiry date may not be returned.

Disclaimer

The recommendations for storage, care, maintenance, reprocessing and sterilization have been carefully checked, conform to international standard ISO 17664 and are considered to be appropriate. The person who actually reprocesses the instruments is responsible for achieving the desired results with the provided equipment, materials and personnel in the reprocessing facility. Any deviation from the instructions in the reprocessing section on page 21 should be analysed and assessed in respect of their new possible risks.

The user is responsible for complications or other negative consequences which may result from reasons such as an incorrect indication or surgical technique, inappropriate material selection, inappropriate application or handling of the instruments, or any kind of application that is not described in the intended uses and any incorrect use section, and cannot be blamed on the manufacturer, importer or supplier of PB Swiss Tools products. No liability is accepted in the event of failure to observe the instructions in this Surgical Technique.

Single-use products

Products intended for single use should never be reused. The reuse or reprocessing of single-use products can harm the product or jointly reprocessed products and/or lead to product failure. Furthermore, the reuse or reprocessing of single-use products increases the risk of contamination, potentially resulting in injury, illness or death of the patient or user.

INSTRUCTIONS FOR SINGLE-USE PRODUCTS

These instructions for use apply to all sterile inserts supplied by PB Swiss Tools GmbH and listed in this Surgical Technique.

- Screwdriver inserts
- Extraction Screws
- Extraction reamers
- Extraction drill bits

Sterile-packed, for single use

Inserts may not be reprocessed or resterilized.

After determining the correct insert size, remove the sterile insert from the packaging using aseptic surgical techniques. Do not resterilize and do not reuse if the packaging is damaged or torn.

Plasma coating

The color-coding of the inserts consists of a plasma coating. Slight deviations in color or shading result from the manufacturing process and do not affect the quality of the inserts.

INSTRUCTIONS FOR REPROCESSABLE PRODUCTS

These instructions for use apply to all reprocessible surgical instruments supplied by PB Swiss Tools GmbH and listed in this Surgical Technique.

- Handles
- Cross-handles
- Extensions
- AO/Synthes couplings

The instruments are supplied in non-sterile form and must be reprocessed before every use. This also applies to the first use of reprocessible surgical instruments after delivery. Sterilization of the delivery packaging is not possible or permitted. Please note that only validated methods may be used for cleaning/disinfection and sterilization. The respective national and any applicable internal regulations must be observed. For further information on cleaning and sterilization see the reprocessing section on page 21.

INTERPRETATION OF SYMBOLS

	Catalogue number
	Lot No./Batch code
f	Serial number
M	Manufacturer
N	Date of manufacture
H	Expiry date
d	Non-sterile
IK	Sterilized by radiation
D	For single use
B	Do not resterilize
L	Do not use if the sterile packaging is damaged
Y	Caution, consult the instruction for use
	Hex screw recess
	Torx®/Stardrive® screw recess
	Square/Robertson screw recess
	Cruciform screw recess
	Slotted screw recess
	Phillips screw recess
	Screw diameter
	Extraction screw
	Extraction drill bit
	Extraction reamer
C₁₂₅₀	CE mark with identification number of the recognized inspection body
	Applying pressure
	Very little pressure
	Turn clockwise
	Turn counterclockwise
	Use manually only
	Cooling
	Slow speed

REPROCESSING

CLEANING AND STERILIZATION INSTRUCTIONS FOR REPROCESSABLE INSTRUMENTS

Limitations associated with reprocessing

The life of the instruments is not unlimited. The maximum number of processing cycles cannot be determined in advance. The life of the instruments is determined by their correct use and by careful handling. The instruments should be meticulously examined before every use in order to ensure that they are fully functional.

Preparation for cleaning / disinfection

The handles, extensions and couplings should be cleaned and disinfected as soon as possible after use. Dried-on contaminants may impede cleaning.

Cleaning / disinfection, rinsing and drying

Since mechanical methods can be standardized and are much more effective, they should be given preference over manual methods.

Mechanical reprocessing

Use only validated washers and disinfectors (WDs) in accordance with ISO 15883. The operating instructions of the WD manufacturers must be observed. Detergents with a pH of 7–9.5 are recommended, subject to a maximum pH of 11. The handles and extensions must be loaded with the holes face down so that the detergent can drain properly.

Typical washer-disinfector program:

1. Prewash for 3 min, cold tap water
2. Add detergent / disinfectant
3. Wash for 10 min at 55 °C
4. Rinse for 1 min with tap water
5. Rinse for 1 min with demineralized water
6. Thermal disinfection (at least 5 min at >90 °C)
7. Dry (according to the drying cycle of the washer/disinfector). If necessary, continue drying cannulations with sterile-filtered compressed air).

Manual reprocessing

Manual cleaning should be done only if the option of mechanical reprocessing is not available. For manual cleaning, the components are placed in a solution of a combined detergent / disinfectant with a proven disinfectant action. The solutions must be prepared according to the manufacturer's instructions (concentration, temperature, contact time). Detergents and disinfectants with a pH of 7–9.5 are recommended, subject to a maximum pH of 11. Pay particular attention to the cleaning of cannulations (e.g. at the front of the coupling).

Typical manual method:

1. Place product in cold water for at least 5 min
2. Remove blood and other residues with a soft brush (do not use metal brushes or scourers)
3. Immerse product for at least 10 min in a bath with detergent / disinfectant at 45 °C or according to the manufacturer's instructions
4. After cleaning, thoroughly rinse the product with demineralized water. Cannulations must be filled and emptied several times
5. Dry the instruments

INSPECTION

Cleanliness

After every cleaning/disinfection process the products must be free of visible residues and other contaminants. The products, and particularly the cannulations on the couplings, should be carefully inspected for possible residues. If necessary, the reprocessing process should be repeated until the product is visually clean.

Function

Before each use the surgical instruments should be checked for fractures, cracks, distortions, corrosion and correct function. On couplings, it must be possible to move the slide ring gently back and forth.

Packing and labelling

Pack the products / instrument tray as soon as possible after reprocessing in a sterilization bag or sterilization container according to ISO 11607 and label the packaging accordingly.

Sterilisation

We recommend steam sterilization for the handles, extensions and couplings. The sterilization must be carried out in validated, properly maintained steam sterilizers according to ISO 17665.

- Steam sterilization in saturated superheated water steam
- Sterilization at 134 °C, holding time 18 minutes

Drying time 20 – 60 minutes depending on the packing material or container and the performance features of the sterilizer. The operating instructions must be followed in all cases.

Storage

The instruments should be stored in a clean, dry environment, protected from direct sunlight, vermin, extreme temperatures and humidity.

Torx® is a registered trademark of Acument Intellectual Properties, LLC, a company of Acument Global Technologies, Inc.
Stardrive® is a registered trademark of Synthes GmbH.
© PB Swiss Tools, Switzerland. All rights reserved.

 **SWISS TOOLS**

Legal Manufacturer:
PB Swiss Tools GmbH
Bahnhofstrasse 24
CH-3457 Wasen
info@pbswisstools.com



Distributed by:
Johnson & Johnson Medical Pty Ltd
t/a DePuy Synthes
1-5 Khartoum Road
North Ryde NSW 2113
AP/TRM/0518/0261 May 2018

This publication is intended for use in Australia only.
Subject to modification. All rights reserved.

DSEM/TRM/1115/0546 REF 80237 EN/Ver F 12/15